

VOXCOM II

Instruction Manual

Enjoy Your Voxcom II

If you have any questions or need assistance
please call :
1-800-522-6294

Distributed by:

REIZEN™

www.maxiaids.com



Item #: 308428

*Thank You for your purchase and
Welcome to the World of VOXCOM II*

A remarkable, Innovative audible labeling device for recording and hearing vital information in any chosen language!


Now you will be able to hear what you have been missing!

This item is ideal for labeling prescriptions, or having medication instructions audibly labeled, Use it to identify food items, canned goods, clothing, books, cd's and many other items to numerous to mention.

When an item needs to be identified, or short instructions heard, simply insert the card and hear what you have been missing - No need for guessing any more. The VOXCOM II was developed with your needs in mind - It is convenient and easy to use!

Note: There are two different units available—one designed for use with 50 cards and another for use with 100 cards.

This unit is powered by a 9 volt battery.

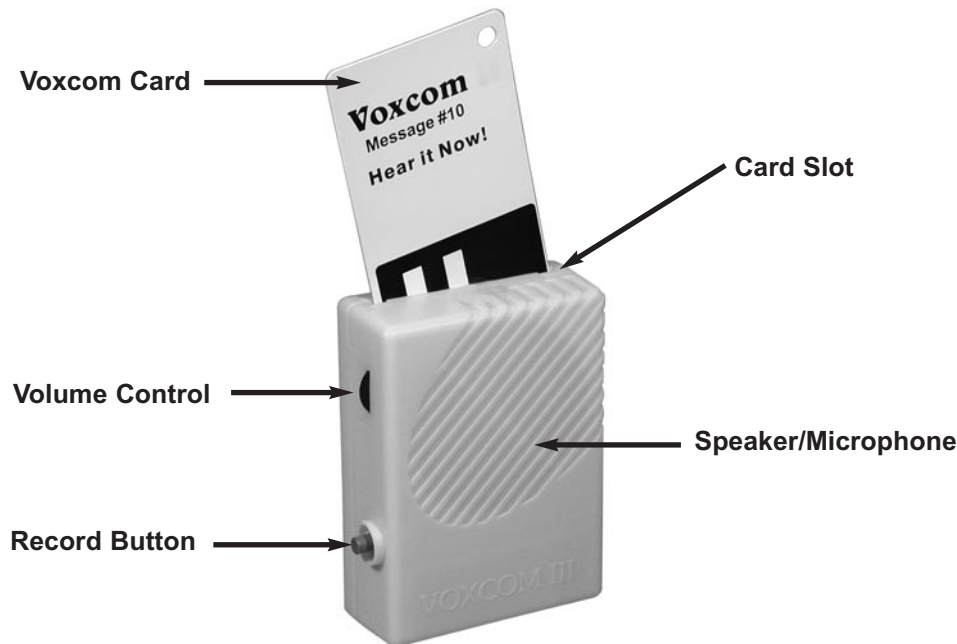

The unit features a microphone and a speaker as well as two buttons on the side of the unit. You should be holding the Voxcom in a position where the speaker/microphone vents are facing you and the 2 buttons are located on the left side of the unit.

Now that you are familiar with the design of the Voxcom you are Ready to operate.

Operating instructions:

Each card is bar coded and numbered - However there is no pre-recorded message on the cards, which indicates the card number.

To Record: Hold the Voxcom approximately 9-12 inches away from your mouth simply press the bottom record button on the Voxcom - while holding the button insert one of the cards into the slot on top of the unit-press card into the Voxcom until you hear or feel a click-now you can begin to record your message. You have approximately 10 second of recording time - per card. When you have completed recording release the card.

To Playback: Simply press the card in the slot until you hear a click (Warning DO NOT HOLD RECORD BUTTON AT THIS TIME) and your message will be played back.

You can record over the card many times so the cards can be reused time and time again.

We have supplied you with several choices for adhering card to product for identification; Rubber bands, Adhesive backed Velcro, Adhesive backed magnets and plastic pull ties.

To Make Louder:

Adjust volume control dial by turning dial downwards to increase volume and upward to decrease volume.

To Change Battery:

Slide down battery cover and remove old battery and replace with new one. Slide back cover until it clicks into place.