



For all the world to see®



HANDS-FREE DUAL LENS MAGNIFIER

Thank you for choosing MagEyes®. Your purchase, properly cared for, will provide many years of valuable service and enjoyment.

USE INSTRUCTIONS:


1. Gently spread ends of headband and place MagEyes around your forehead like a visor. May be worn with prescription glasses.
2. Loosen turn-knobs at temples, if necessary. Position MagEyes lenses directly in the line of vision and tighten turn-knobs to lock the lens in position.
3. Adjust focus and enlargement by moving the object you are viewing closer to or farther away from the lenses.
4. Pop lenses out to interchange. Flat side should be towards your face.
5. **PRECAUTIONS: Accidental combustion could take place if the lenses are left lying in direct sunlight. DO NOT look directly into the sun through the lenses. MagEyes is intended as a vision aid and NOT as a replacement for prescription eyewear. MagEyes is not intended to provide eye protection from liquids, gasses or flying objects. If your occupational or recreational activities exposes you to severe risk of flying objects or physical impacts, your eye safety may require special safety spectacles with safety lenses, side shields, goggles, and/or a full face shield.**
6. If foam comes loose (due to heat) from headband, press back on with fingers.
7. All lenses are conveniently packaged individually.



1.



3.

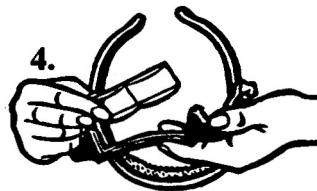

4.

Pull-apart

U.S. and Foreign Patents Pending

5548841

359060

5727251

Canadian (D) 75241

CLEANING AND CARE INSTRUCTIONS:

1. Use a solution of mild soap and warm water only.
2. Rinse in cool, clear water.
3. Gently press water out of all cushioned parts. Wipe all other parts dry with a soft optical cleaning cloth. Do not use paper products, cleaning solvents or abrasives to clean lenses.
4. MagEyes is intended as a vision aid and not as a replacement for prescription eye wear. MagEyes is not intended to provide eye protection from liquids, gasses or flying objects. If your occupational or recreational activities exposes you to severe risk of flying objects or physical impacts, your eye safety may require special safety spectacles with safety lenses, side shields, goggles, and/or a full face shield. **Do not wear in direct sunlight.**

LIMITED WARRANTY

This quality MagEyes Magnifier is warranted against defects in material and workmanship for a period of 90 days from date of original purchase. **EXCEPT AS HEREIN PROVIDED, NO WARRANTY OF ANY KIND, EXPRESS OR IMPLIED IS MADE AND ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, ARE HEREBY DISCLAIMED.** Should a defect occur, simply return the MagEye to your original point of purchase and include your receipt.

WARNING:

**Choking Hazard - Small parts.
NOT for children under 3 years of age.**

*All parts manufactured and
assembled in the U.S. of A.*