

TALKING WATCH INSTRUCTIONS

1-BUTTON ANALOG TALKING WATCH


1. Crown-Adjust hour and minute hands.
2. S1-Talking voice function, press down to activate voice chip to speak time.

SET HOUR AND MINUTE HANDS IN TIME:

- Pull out the crown and scroll the crown to set hour and minute hands until the desired time have been set. Then push back the crown to normal position.

SET TALKING VOICE TIME PROCEDURES:

- Use a pin to press S2 to set hour time.
- Use a pin to press S3 to set minute time and all is now done.

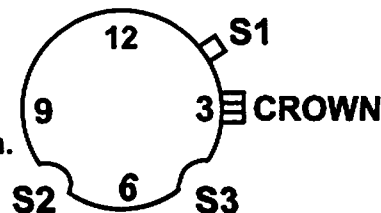

TALKING VOICE QUARTZ ANALOG ALARM WATCH

(I) BASIC FUNCTION FOR EVERY PUSH BUTTON AND CROWN:

1. Crown adjust hour and minute hands.
2. S1-Talking voice function. Press down to activate voice chip to speak time.

Not for use with
Calendar/Timer Watch
(See Reverse)

3. S2 - A) "Chime off" and "chime" function:

- When "chime" is on it activates voice to tell time every hour.


- B) Another function: after pushing S3 to the "set time" or "set alarm time" mode. Each push of the S2 button advances voice time one minute.

4. S3 - Set mode: A)time B) set time C)set alarm time

5. S4 - A) Set "alarm off"

- B) Set "alarm on" : by following choice of sound
 - : "rooster" sound alarm
 - : "bi bi" sound alarm
 - : "dong dong" sound alarm

- C) Another function: after pushing S3 to the "set time" or "set alarm time" mode, each push of the S4 button advances voice time one hour.

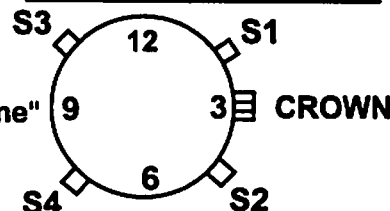

(II) INSTRUCTION:

1. SET HOUR AND MINUTE HANDS IN TIME:

- Pull out the crown and set hour and minute to correct time. Then push back the crown to normal position.

2. SET TALKING VOICE TIME PROCEDURES:

- Press S3 set mode button to "set time" mode.
- Press S4 to set hour time.
- Then press S2 to set minute time. (always allow 15 sec. time to set the minute time to match minute hands before sweep hand touches 12: otherwise, that would create a minute difference between minute hands and voice time.)
- Press S3 back to "time" mode for normal use and all is done.
- Press S1: talking voice will have same time as hour & minute hands shown.

3. SET ALARM VOICE TIME PROCEDURES:

- Press S3 set mode button to "set alarm time" mode.
- Press S4 to set hour time.
- Then press S2 to set minute time and all is done.
- Press S3 back to "time" mode for normal use.
- Press S4 to select any of 3 alarm sound as alarm on function:
 - either "rooster" or "bi bi" or "dong dong"
- Press S4 to alarm off - that would turn off alarm function.

4. TO CHECK EXISTING ALARM AND CHANGE ALARM TIME:

- A) Press S3 set mode button to "set alarm time" mode then press S1 button to hear previous existing alarm time.

- B) To change alarm time, follow #3 instruction.

5. SET CHIME ON/OFF PROCEDURES:

- Press S2 alternately to activate on and off voice function to speak the time every hour.