

7 ALARMS

1. Press the **LEFT** button & release, causing the “alarm 1” symbol to start blinking. Set the alarm **hour** using the **MIDDLE** button and **minute** using the **RIGHT** button.



ALARM SET

HOUR

MINUTE

To cancel the alarm, press the **CENTER** button until this image appears:



2. Once you set the correct alarm time, press the **LEFT** button to move onto the next alarm.
3. Once the “alarm 2” symbol is blinking, set this alarm in the same way.
4. Continue setting alarms 3-6 in this way.
5. End by pressing the **LEFT** button until the alarm symbols are no longer blinking.

Hold the **CENTER** or **RIGHT** buttons to automatically progress hour or minutes as desired. Display will return to normal function if left alone for more than one minute.

8 TONE OPTIONS

Choose between alarm tones A, B, or C and your choice will be displayed on the screen. If you choose silent alarms, no choice will be displayed. To change the tone, hold down the **RIGHT** button while pressing the **LEFT** button to cycle through tones.



Once your desired alarm tone is displayed on the screen, release both buttons.

9 TEST

To enter test mode, hold the **CENTER** button until “CH” appears on the screen. To exit test mode, press the **RIGHT** button. In order to perform tests, press the **LEFT** button and the unit will cycle through the following test modes:



- CH1:** All segments on the display will light up.
 - CH2:** Motor rotates the tray by one compartment.
 - CH3:** Alarm will sound and LED lights will light up.
- Following this test, turn the dispenser over to test the “RESET” function. Alarm and LED will stop. If not stopped in 10 seconds, “CH4” will be displayed on screen.

NOTE: If dispenser does not rotate properly, error message “CH6” will appear on the screen. If this happens, press the right button and replace the batteries to reset.

10 LOCKING & UNLOCKING

Insert the key and turn it ¼ to the **RIGHT** to unlock or ¼ to the **LEFT** to lock.



11 STOP ALARM

When the alarm goes off, pick up the dispenser and tilt the pills into your hand or a suitable container.



12 STANDBY/LOW BATTERY

When not in use, the display will shut off and the dispenser enters “Standby Mode.” To wake it up, press the **RIGHT** button.

When batteries are low, a battery symbol with an “X” through it will appear on the screen. When you see this symbol, batteries are low and need to be replaced.

© 2021 Reizen, Inc. All Rights reserved; contents contained within are the exclusive property of Reizen, and its affiliated companies, and may NOT be reproduced without the written consent

REIZEN™
AUT+PILL
USER GUIDE



1**TO START**

Place 4 **AA** batteries into the battery compartment

**2****OPENING THE TRAY:**

Push on front tab with thumb as shown.

**3****CLEANING**

Surface clean with damp sponge or cloth only.

Remove tray after this step.




PLEASE NOTE: **do not** place product in direct sunlight or strong ultraviolet light to avoid malfunction

4**FILLING DISPENSER**

Dispenser includes 6 discs labelled with each day of the week and the number of doses taken each day (1-6 times).

1. Once a day
2. Twice a day
3. Three times a day
4. Four times a day
5. Five times a day
6. Six times a day

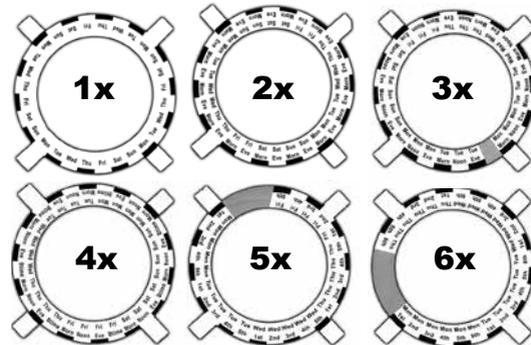

With clear plastic tray removed, place the desired disk around hub as shown then fold down tabs to secure it.



Then return clear plastic tray back into place.

Organize your medications and assign dose times by yourself, with a caregiver, or with the help of a healthcare professional.

Depending on how many doses of medication you take per day, you may need to refill your dispenser more quickly.

- 1x/day: every 28 days
- 2x/day: every 14 days
- 3x/day: every 9 days
- 4x/day: every 7 days
- 5x/day: every 5 days
- 6x/day: every 4 days

**5****CHANGING TIME FORMAT**

Dispenser defaults to 24-hour time format. To switch to a 12-hour format, **press buttons 2 & 3 at the same time** until an AM or PM symbol appears on the right side of the display.

When your preferred time format is displayed, release the buttons.

**6****SETTING THE TIME**

1. Press & hold **left** button until “:” is blinking, then release.



2. Set the hour using the **middle** button. Set the minutes using the **right** button.

**HOUR****MINUTE**

3. confirm time by pressing **left** button again.

