

Royal Tel-Weight

Digital Talking Bathroom Scale (ITEM # 203778)
Bilingual : English – Spanish

Owner's Manual

I. Before Using

Insert four AA batteries into the battery compartment at the back of the scale. Use the language switch on the bottom side to toggle between English and Spanish. Place the scale on hard, level ground.

Please balance yourself properly for safety.

II. Taking Measurement

- 1) Use one foot to tap on the measurement platform with at least five pounds of force. When it says “Hello!”- remove your foot from the platform.
- 2) When it says “I’m ready!” – step onto the platform. Your weight will be read out once you remain steady.
- 3) The scale will turn off within 10 seconds after you step off the platform. It’ll say “Goodbye!” when it powers off.

III. Switching to the Metric System

Press the LB/KG button below the display panel near the top of the scale to switch between metric system (KG) and non-metric (LB) readings.

IV. Turning off the Talking Function

Use the switch at the bottom of the platform to switch on/off the talking function.

V. Language Conversion

Use the language switch at the bottom of the platform to toggle between English and Spanish. Language change will take effect after the scale is off and then re-activated.

IMPORTANT NOTES

- Always balance yourself for safety. When taking measurement, be sure you are evenly balanced on the measurement platform.
- When overloaded, “FFFF” will be displayed.
- When the batteries are running low, the warning icon will be displayed. When this happens, please replace the batteries immediately.
- Use the unit on hard, flat and secure surface only.
- Place the unit in an area free from high temperature or high humidity. To clean, use dampened cloth and mild detergent. **DO NOT** submerge the unit in water.
- Avoid excessive impact or vibration to the unit.
- If you do not intend to use this unit for a long period of time, remove the batteries before storing.
- Do not attempt to disassemble the unit, or the warranty will become void.
- The unit is intended for home and personal use only. **Not** for commercial use.