

I N S T R U C T I O N S



Steady Write® Sta-Pen™ Writing Instrument


Patent No. 3,972,628

Catalog Number 73508-1000

The Steady Write® Sta-Pen™ Writing Instrument uses the friction of the base to smooth out the motion of your writing.

For best results follow these easy steps:

1. Place the Steady Write® Sta-Pen™ base on the writing surface.
2. Grasp the Steady Write® Sta-Pen™ base (not the pen) and press down, moving the whole unit as you write.

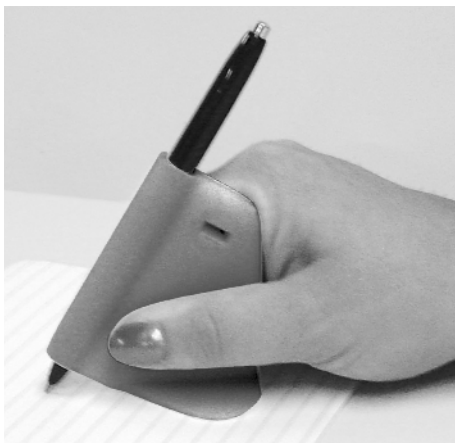

Note: You may have to experiment with different ways of grasping the base to find the best method for you.

For best results:

Do not hold any part of the pen. Press down on the base.

Do not raise the base off the writing surface as you write.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com