
## Mist XP™ Humidifier

Model: 40-685-000

**Instructions for Use**

Please read these instructions carefully and keep them for future reference.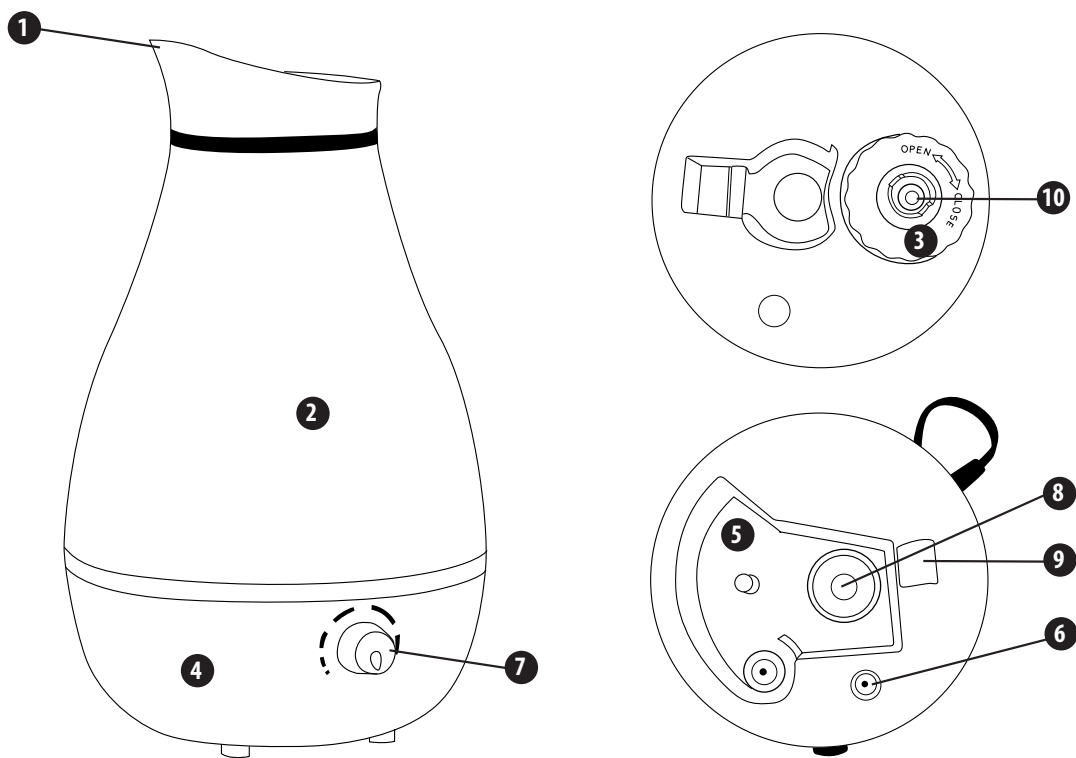

### **KEY TO SYMBOLS**

---

- 1) Mist Output Opening
- 2) Water Tank
- 3) Tank Cap
- 4) Base
- 5) Reservoir

- 6) Nightlight
- 7) Power Knob / Adjustable Mist Knob
- 8) Ultrasonic Transducer
- 9) Air Outlet
- 10) Water Valve

## ***Important Safety Instructions***

---

- After removing all packaging materials, ensure that the humidifier has not been damaged during transportation.
- Do not leave any packaging materials (plastic bags, cardboard boxes, etc.) within the reach of children to avoid injury.
- Before connecting the equipment, ensure that the rated voltage (120 V) corresponds to the main voltage supply of the electrical outlet.
- Do not use additional power strip adapters, and/or extension cords to avoid the risk of fire, shock or injury.
- Only place the humidifier on flat and stable surfaces.
- Only use clean, cool tap, filtered or distilled water to fill the water tank. Do not use hot water. Do not add salt, fragrance/essential oils, medication or descaling tablets to the water.
- Do not operate on wood or other water sensitive surfaces, as condensation may occur.
- Do not leave the humidifier unattended in a closed room during use, as the air could become too saturated and leave condensation on furniture or walls.
- Always disconnect the equipment from the power outlet when it is left unattended or not in use.
- Before performing any cleaning or maintenance, disconnect the equipment from the power outlet.
- This product is not a toy. Keep out of reach of children.
- This equipment is a household humidifier for indoor use only. Do not operate outdoors.
- Do not use accessories, spare parts, or any component not supplied or approved by the manufacturer.

### ***Important Safety Instructions (continued)***

---

- Never touch the equipment with wet or damp hands.
- Do not move, tilt or tip the humidifier during use. Doing so may cause water leakage.
- If the equipment is damaged and/or not working properly, turn the unit off by disconnecting the plug from the power outlet.
- Never submerge the equipment in water or any other liquid.
- Do not touch the equipment if it accidentally falls into the water.
- Do not direct the vapor flow towards furniture or electrical appliances.
- Do not touch the ultrasonic transducer with your hands or tools while the unit is plugged in.
- Do not operate the humidifier without water.
- Do not attempt to empty or fill the tank while the unit is plugged in.
- Clean and dry the humidifier at the end of every operating cycle.
- Dispose of any worn accessories and/or the equipment at an appropriate collection point, as required by state regulations.

### ***How to Prepare and Use the Equipment:***

---

- Check that the equipment is completely clean, the power knob is in the OFF position and that the power cord is not connected to the power outlet.
- Gently remove the water tank from the base.
- Unscrew the cap on the underside of the tank, and fill the water tank, leaving approximately 1/4" of space from the rim when the tank is upside down.  
Do not fill water all the way to the rim.
- Screw the cap back on, and fit the water tank right side up back into the base of the unit, making sure it is properly secured in place.
- Connect the equipment to a power outlet.
- Use the mist regulation knob to turn the unit ON by moving the knob clockwise.
- The nightlight illuminates when the unit is ON, and vapor will start to exit.
- Use the mist regulation knob to control the amount of vapor flow desired.
- To turn the unit OFF, turn the knob counterclockwise. The nightlight will turn off when the unit is OFF.
- When the water tank is empty, the humidifier stops automatically.

## ***Cleaning and Maintenance***

---

- Before cleaning the equipment, always turn the humidifier OFF, and disconnect the plug from the power outlet.
- Do not place the humidifier directly under running water.
- Do not wet the power cable or air outlet.
- Do not scrape the ultrasonic transducer with metallic materials, tools or objects.
- Do not leave water in the water tank when the equipment is not in use.

### **Daily Cleaning:**

- Remove water tank from the base, unscrew the cap and pour out all remaining water.
- Clean the tank, reservoir and transducer of the humidifier with a soft, clean cloth and dry thoroughly.
- Dry all parts thoroughly before storage.

### **Weekly Cleaning:**

- Remove water tank from the base, unscrew the cap and pour out all remaining water.
- Pour a water solution prepared with 50% white vinegar into the water tank.
- Screw the cap back on, and fit the water tank right side up back into the base of the unit, making sure it is properly secured in place.
- Leave soak for at least 4 hours (overnight is ideal). Do not turn the humidifier on and run the unit with the water and vinegar cleaning solution.
- Remove the water tank and cap to pour out all of the water/white vinegar solution, and rinse the tank with cool, clean water.
- Wipe the tank, reservoir and transducer of the humidifier with a soft, clean cloth and dry thoroughly.
- Clean the outer surfaces of the humidifier with a soft, damp cloth.
- Do not use abrasive products, alcohol, solvents or similar products which may cause permanent damage to the outer surface.

## ***Troubleshooting***

---

<b>Problem</b>	<b>Solution</b>
If the humidifier does not work	<ul style="list-style-type: none"><li>• Check that the plug is connected correctly.</li><li>• Check that the switch is in the ON position.</li><li>• Check that there is sufficient water in the water tank.</li><li>• Check that the water tank is properly in place and secured on the base.</li></ul>
If the power button is in the ON position but there is no vapor flow	<ul style="list-style-type: none"><li>• Check whether there are hard water deposits on the ultrasonic transducer.</li><li>• Check whether there is sufficient water in the water tank.</li></ul>
Strange odor	<ul style="list-style-type: none"><li>• Clean the water tank and the reservoir with water and vinegar solution, then rinse with clean water.</li><li>• Aerate the water tank in a cool place for 12 hours.</li></ul>

## ***Technical Specifications***

---

Run Time: Min. up to 24 hours / Max. up to 10 hours

Tank Capacity: 0.66 gal.

Ideal Room Size: Medium to Large

Average Humidity Output\*: 7.1 fl. oz./hr

Power Consumption: 120V, 60Hz, 22W

Sound Level: <35 dBA

\* Humidity output may vary based on indoor room temperature, humidity and mist output setting.

## Limited Three-Year Warranty

Your Mist XP™ Ultrasonic Cool Mist Humidifier is guaranteed to be free of manufacturing defects for a period of three years from the date of purchase under normal use, certain exclusions apply. This limited warranty does not apply to use in any clinical or commercial applications. If the unit fails to operate during the three year period of the original registered purchaser, please call our Customer Care Help Line at 800-622-4714 and we will try to resolve the issue as soon as possible. If Briggs Healthcare® determines that the unit failed to operate due to a manufacturing defect, the unit will be replaced at the option of Briggs Healthcare®. Replacement of the unit is the sole remedy under this limited warranty. This warranty gives you specific legal rights, which vary from state to state.

This Limited Warranty constitutes Briggs Healthcare's only responsibility and obligation to replace the unit. We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use or purpose has been disclosed to Briggs Healthcare® in specifications, drawings or otherwise, and whether or not Briggs Healthcare's products are specifically designed and/or manufactured by Briggs Healthcare® for the buyer's use or purposes, except for the limited warranty stated above. Briggs will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. Briggs shall in no event be liable to the purchaser for any amount in excess of the cost of replacement of the unit.

Intertek

CONFORMS TO UL STD 998

CERTIFIED TO CSA STD. C22.2 NO. 104

This product has been tested and complies with the requirements for Federal Communications Commission, Part 18 for Radio/TV/Communication interference. Although tested, it may affect these devices. If the humidifier is found to interfere, separate the device and/or the humidifier. Conduct only the user maintenance found in this manual. Other maintenance and servicing can cause harmful interference and can void the required FCC compliance.

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©2013 Briggs Healthcare