


# XL-550

HIGH CAPACITY (TALKING SCALE)

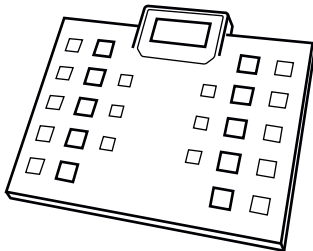

ENGLISH



Capacity  
250kg x 0.1kg  
550lb x 0.2lb

Thank you for purchasing the My Weigh® XL-550 Talking digital scale. Please read all operating instructions carefully before use. This electronic scale is a precision instrument. With normal care and proper treatment, it will provide years of reliable service. For more information please visit [www.myweigh.com](http://www.myweigh.com)

## OPERATION INSTRUCTIONS

### Inserting batteries

Insert 4 AA batteries into the battery compartment on the bottom (underside) of the scale. Be sure the polarity (+ / -) is correct when installing batteries

### Taking measurement

- Step onto the measurement platform. The scale will say "Hello". Now step off the platform and wait a few moments
- Once the scale is stable, it'll say "It's ready" and show "0.0 lb" (or 0.0 kg). Then you may step onto the measurement platform again. Your weight will be displayed and spoken once the reading is steady.
- You may step off of the platform and the reading will hold for 10 secs. After that it'll say "Good-bye" and turn off automatically.

### Power off

The power will shut down automatically after no operation is detected for about 10 seconds.

### Switching on/off the talking function

Use the switch on the bottom of the platform to switch on/off the speaker function.

### Unit conversion

Turn the Scale ON, "0.0" will show on the display" press the [kg/lb] button on the front of the scale (under the display) to select the desired unit (LB or KG)

### Selecting language

After showing "0.0", press and hold the [kg/lb] button for 7 seconds. You will hear the word "Hello" spoken in the current language. Press the [kg/lb] button to toggle through the available languages. Once you have selected the desired language confirm the selection by pressing and holding the [kg/lb] button until the scale says "Goodbye".

## IMPORTANT NOTES

- \*\*\*If the Scale ever malfunctions please try installing a new battery, sometimes a low battery or defective battery may cause inaccurate readings or other errors
- When the battery runs very low, the warning icon will appear on the display. When this happens, be sure to replace the battery immediately on the underside of the scale.
- When over-loaded, "FFFF" will be displayed.
- Use the unit on hard, flat, dry and secure surface only.
- Place the unit in an area free from high temperature or high humidity. To clean, use dampened cloth and mild detergent.
- Avoid excessive impact or vibration to the unit.
- If you do not intend to use this unit for a long period of time, remove the battery before storage.
- Do not attempt to disassemble the unit, otherwise the warranty will become void.
- The unit is intended for home and personal use only (not legal for commerce).
- This scale must only be used on a hard surface such as tile, wood, or linoleum
- This and most bath scales will not perform properly on thick-carpeted surfaces.

<b>Capacity</b>	250kg x 0.1kg 550lb x 0.2lb	<b>Units</b>	kg, lbs
<b>Deviation</b>	+/- 1%		
<b>Temperature range</b>	Optimum 15-20°C		
<b>Power source</b>	4 x AA alkaline batteries		

