
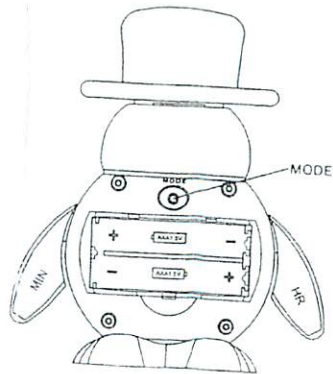


700798

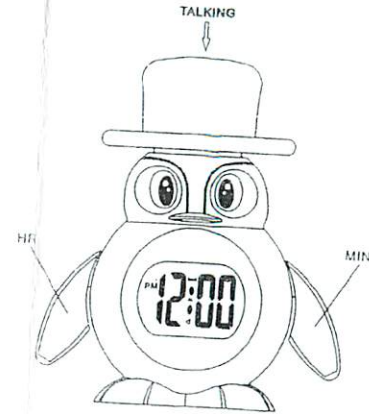

## OPERATING INSTRUCTIONS

700798

## Operating instruction

### Functions description


Talking time, alarm with choice of sound, snooze, hourly chime (two types).

### Battery

Open the battery cover and put two new AAA batteries to right position.

### Mode

Press MODE to change the functional modes according to the following direction:

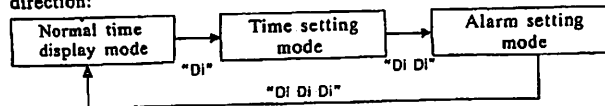

### To set time

Starting from time display mode, press MODE to enter time setting mode. Press MIN, HR to adjust minutes and hours respectively to the appropriate alarm time.

### To set Alarm time

Starting from time display mode, press MODE twice to enter alarm setting mode. AL symbol will display with blinking numerals. Press MIN, HR to adjust minutes and hours respectively to the appropriate alarm time.

### Choose the alarm sound and turn on or off the alarm

In normal time display, press HR button to choose the alarm sound and turn on or off the alarm. When setting with alarm sound, it will play the alarm sound and the alarm time is shown on the display, the alarm function is on, after playing the alarm sound, it returns to the

normal time display. When setting without alarm sound, alarm function is off, then it plays "Di".

### Turn on/off hourly announcement

Starting from time display mode, press MIN to activate (see symbol) or deactivate hourly announcement (no symbol).

### Snooze function

When the alarm sounds, press the button on top of the clock to stop the alarm and start the snooze function. The alarm will sound again every five minutes after the snooze starts. The alarm sound will happen not more than six times.

### Listen to the alarm sound

In normal time display mode, press and hold top of the clock. It will play the alarm sound after announcing the current time.

### Remarks

Avoid exposing the clock to extreme temperature, water or severe shock. Avoid contacting with chemical medicine.

Don't subject the clock to high temperature, excessive humidity, dust, shock.

Don't use old batteries or mix old and new one.

Only professionals can repair the clock.

3 month Limited Warranty.

Distributed by REIZEN  
P.O. Box 3312  
Farmingdale, N.Y. 11735  
[www.ReizenUSA.com](http://www.ReizenUSA.com)