

Congratulations on your purchase of the All Terrain Cane!

The All Terrain Cane was designed to help the blind navigate more safely and easily in areas that aren't paved. The long curved tip reduces snagging, allowing it to glide through sand, grass and other rough terrain more easily. Now you'll be able to walk with confidence on the beach or in the park!

Plus, you can convert your All Terrain Cane to a standard cane by simply changing the tip to a Bulldog Rolling Cane Tip (see 'OPTIONAL ITEMS' section below)

ASSEMBLING YOUR CANE:

RIGID CANE: Your cane will come in two pieces – (a) the aluminum shaft and (b) the curved nylon cane tip piece with a double bracket and set screws.

To assemble the cane:

1. Locate the set screw closest to the end on the cane tip piece
2. Simply slide the cane tip over the end of the aluminum shaft (Note: It may be necessary to loosen the set screw so the cane fits into the bracket)
3. Tighten the set screw until the cane tip piece is attached firmly to the shaft

HOW TO USE YOUR CANE:

Hold the cane out ahead of you resting on the ground. As you walk, swivel your wrist from left to right, so that the tip moves along the ground in an S pattern, much like the movement of a snake. The curved tip should remain on the ground at all times.

OPTIONAL ITEMS:

Interchangeable Rolling Cane Tips:

Your All Terrain Cane may be quickly and easily converted to a traditional cane for use on paved sidewalks and roads. Simply loosen the thumbscrew at the base of the cane, remove the All Terrain Tip, and replace it with one of these optional Rolling Tips:

Bulldog Hook Type Rolling Cane Tip: White – Large (Item #172833)

Bulldog Hook Type Rolling Cane Tip: White – Small (Item #172843)

[Please note that with the Bulldog Tip is attached, the cane will be approximately 3 inches shorter than it is with the curved All Terrain Tip attached.]

Visit www.MaxiAids.com or call 1-800-522-6294 for more information