

**“READ ALL INSTRUCTIONS BEFORE USING”
IMPORTANT SAFEGUARDS**

DANGER - to reduce the risk of electrocution:

1. Always unplug this product immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid

Warning - To reduce the risk of burns, electrocution, fire, or injury to persons.

1. A product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been damaged, or dropped into water. Return the product to the manufacturer for examination and repair.
5. Keep the cord away from heated surfaces.
6. Never use while sleeping or drowsy.
7. Do not use where aerosol (spray) products are being used or where oxygen is being administered.
8. **Do not get the syringe end near eyes or ears, as the needle can puncture skin.**
9. **This product is intended to clean hearing aids, any unauthorized use will void any and all warranties.**

“CAUTION” “RISK OF ELECTRIC SHOCK, DO NOT REMOVE COVER. REFER SERVICING TO QUALIFIED PERSONNEL”

This product is for use on a nominal 120 V circuit. This product is factory equipped with a specific electric cord and plug to permit connection to a proper electric circuit. Make sure that the product is connected to an outlet having the same configuration as the plug. Do not modify the plug provided - if it will not fit the outlet, have the proper outlet installed by a qualified electrician. If the product must be reconnected for use on a different type of electrical circuit, the reconnection should be made by qualified service personnel.

WARRANTY

Manufacturer will repair or replace the product for all defects and workmanship for a period of 12 months from the date of purchase.

Syringe assembly and needle are excluded from this or any other warranty.

Manufacturer is not responsible for incidental or consequential damages arising from accident abuse, misuse or failure to follow instructions for the product. Any alteration of the product voids this warranty.

To activate this warranty, purchaser must send in the warranty registration card and proof of purchase of the product.

Purchaser is responsible for shipping and handling costs when returning product for warranty service. For warranty service contact;

Advanced Hearing Technologies, LLC

PO Box 18232

4410 NE Mason St.

Portland, OR 97218-0232 USA

Toll Free 1-866-856-5634

voice 503-284-1906

fax 503-287-8408


www.jodivac.com

jodi User Instructional Brochure

Advanced Hearing Technologies, LLC
PO Box 18232
Portland, OR 97218-0232
voice 503-284-1906 USA
fax 503-287-8408 USA
www.jodivac.com



1.* Insert plug end of power transformer into box.

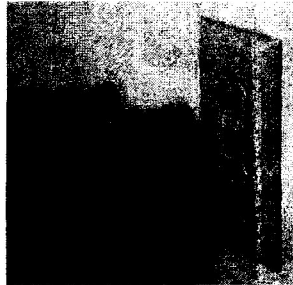


2.* Plug Transformer into a 120volt AC Outlet



3.* The circle on the switch indicates OFF
The line on the switch indicates ON





A


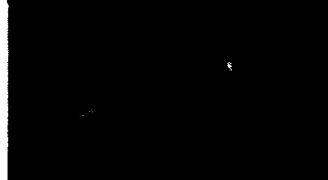


B


4.* A: With jodi turned on, normal operation, the ball will travel to the blue tubing.
B: When the ball falls towards the filter;
* The needle is plugged
* The needle fits too tight into the aid
* The needle is inside the receiver or microphone
* The syringe is plugged

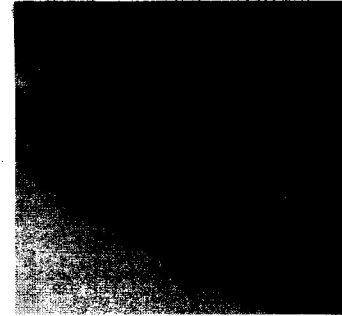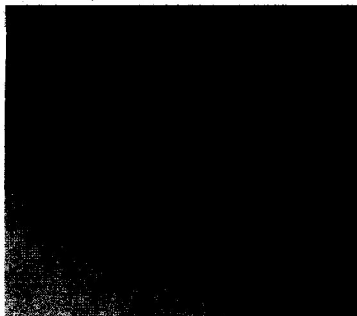

5.* Work slowly allowing the needle time to clean the receiver and microphone tubing on the way in. DO NOT USE PRESSURE WHEN VACUUMING, YOU MAY DISLODGE A TUBE OR COMPONENT CAUSING DAMAGE TO YOUR HEARING AID.

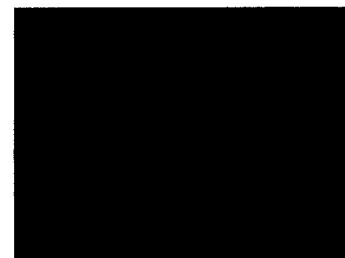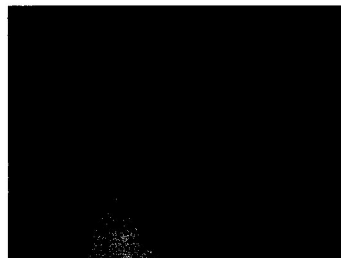

6.* To clean a plugged needle;
While the motor is running, jab the needle into the brush.
If the brush does not clear the needle, while the motor is running, take the reamer and push it through the needle, this will evacuate all of the debris.

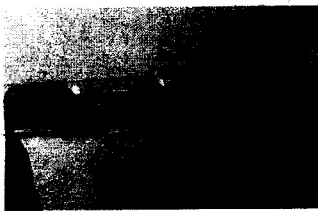

7.* When the syringe is full of debris, disconnect the blue tube from the syringe.
Take the ball and place it into the replacement syringe.
Replace the tubing back into the syringe.
Unscrew the needle (counter clockwise) and attach it finger tight to the new syringe.